Tedequim

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**BIOCERAMIC ROOT CANAL SEALER.
READY TO USE.
BASED ON CALCIUM SILICATE TECHNOLOGY**

Description

Premixed bioceramic endodontic sealing material based on calcium silicates and a radiopaque agent, indicated for the permanent obturation of the root canal system with gutta-percha cones.

Sets in the presence of moisture from the dentin, forming a solid structure with bioactive properties.

Indications:

Indicated for the permanent obturation of root canals in pulpitis, pulp necrosis, or endodontic retreatment, using single-cone or multiple-cone technique with cold lateral compaction.

Properties:

- Allows easy partial removal for post-endodontic rehabilitation.
- Biocompatible.
- Alkaline pH (bactericidal).
- Bioactive: releases calcium ions, forms hydroxyapatite on the dentin surface, and produces a chemical bond.
- Stimulates tissue healing and repair.
- Applied directly from the premixed syringe, with no prior preparation required.
- Good three-dimensional seal due to setting expansion, preventing leakage.
- Moisture-dependent setting.
- High flowability.
- High radiopacity.
- Does not stain tooth structure.
- Contains no resins.

Instructions for use:

1. Apply anesthesia as needed.
2. Isolate with rubber dam.
3. Perform endodontic access.
4. Instrument the root canal.
5. Irrigate and disinfect according to clinical protocol.

Drying

6. Dry with absorbent paper points. Avoid excessive drying.

Application

7. Place an intracanal tip on the syringe.
8. Insert the tip to 2–3 mm short of the working length.
9. Inject the sealer slowly while withdrawing the tip (retrograde technique), without applying excessive pressure, avoiding the tip becoming lodged in the canal.

Obturation

10. Insert the gutta-percha cone to the working length and cut off the excess with a cherry-red heated instrument or an ultrasonic tip.
11. Complete the obturation using the selected technique.

Contraindications:

- Patients with known hypersensitivity to any of the components.
- Do not use in primary (deciduous) teeth, except at the professional's informed discretion.

Warnings:

- Product for exclusive use by dental professionals.
- Avoid extrusion beyond the root apex.
- Do not ingest.
- Keep out of the reach of children.

Precautions:

- Use absolute isolation.
- Do not over-dry the root canal (residual moisture promotes setting).
- Verify canal patency before application.
- Do not reuse applicator tips.
- Dispose of waste properly according to local regulations.

Composition:

Tricalcium silicate, dicalcium silicate, silicon oxide, zirconium oxide, calcium ions, biocompatible hydrophilic polymers.

Contents:

- Syringe x 1 g
- Disposable intracanal tips
- Protect from excessive moisture.

Storage and conservation:

- Store between 4°C and 30°C (39°F-86°F).
- Keep in the original container, tightly closed, with the silica gel packet.

Technical direction: Miryam Della Vedova — Pharmacist — Reg. No. 2107

Authorized by ANMAT PM 1277-35

Exclusive sale to healthcare professionals and institutions.

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