Tedequim

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FAgamin® FLUORURO DE DIAMINO PLATA 38%

PÁGINA..... 2

PÁGINA..... 3

PÁGINA..... 4

INSTRUCTIONS FOR USE

FAgamin® SILVER DIAMINE FLUORIDE 38%

PAGE..... 5

PAGE..... 6

PAGE..... 7

FAgamin®
CARIOSTÁTICO, BACTERICIDA, REMINERALIZADOR Y
DESENSIBILIZANTE DENTAL
FLUORURO DE DIAMINO PLATA

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FAgamin®
CARIES-ARRESTING, BACTERICIDAL, REMINERALIZING, AND
DENTAL DESENSITIZING AGENT
SILVER DIAMINE FLUORIDE

Description and Use:

FAgamin® is a 38% solution of silver diamine fluoride ($\text{Ag}(\text{NH}_3)_2\text{F}$). Its mechanism of action is based on the reaction with the tooth's hydroxyapatite, forming insoluble deposits in the enamel and dentin. These deposits allow the occlusion of dentinal tubules by particles containing silver, calcium, phosphate, and fluoride, reducing their diameter and blocking stimulus transmission, which decreases dental sensitivity.

Additionally, these deposits reinforce the tooth surface, promoting remineralization.

- The silver ion (Ag^+) acts as a bactericidal agent and exhibits anti-enzymatic properties.
- Ammonia stabilizes silver ions, keeping them in solution with a basic pH.
- The fluoride ion decreases hydroxyapatite solubility and inhibits caries progression.
- The high concentration of active ions helps prevent biofilm formation.

Thanks to its composition, FAgamin® enables minimally invasive dentistry through fast, painless topical application without anesthesia. It is indicated for:

- Caries in primary teeth.
- Enamel and dentin remineralization.
- Detection and inhibition of incipient or advanced carious lesions.
- Dentin hypersensitivity.

Clinical Applications:

- Primary teeth with rampant or incipient caries.
- Pits and fissures (previous widening if the professional deems it necessary).
- Cavity preparations for prostheses, crowns, core build-ups, and fixed bridges.
- Root canals (dilute 1:10 with distilled water).
- Patients with severe cognitive or motor disabilities, or dental phobia.
- White spots due to demineralization (early caries or fluorosis).

Contraindications:

Do not use in patients with:

- 1- Ulcerative gingivitis.
- 2- Hypersensitivity to any of the components.
- 3- Recent complete gingivectomy.

Instructions for Use:

1- Isolation:

Protect mucosa using cotton rolls, rubber dam, petroleum jelly, cocoa butter, gingival barrier, or TEDEQUIM® gingival/lip protector.

2- Preparation:

Clean and dry the tooth before application.

3- Dosage:

Dispense 2–3 drops of FAgamin® into a disposable container (not glass).

4- Application:

Apply with a microbrush or cotton swab, gently rubbing the treated area for a few minutes. Remove excess. Do not rinse.

5- Frequency:

- For incipient caries, a single application is often sufficient.
- In other cases, repeat after one week if necessary.
- For desensitization, do not apply to more than 5 sites per session.

Warnings:

- For topical and local use only.
- Do not ingest.
- Keep out of reach of children.
- Protect the patient's eyes and clothing.
- In case of eye contact, rinse thoroughly with water and seek medical attention.
- In case of skin contact, wash with water.
- Avoid contact with gingiva. If contact occurs, rinse with saline or 10-volume hydrogen peroxide.
- May stain soft dentin, restoration margins, or areas with bacterial presence.

Precautions:

FAgamin® generally does not stain enamel but may darken soft dentin or margins of composite restorations. Discoloration may also occur where silver ions react with bacteria. Staining can be minimized with a 5% potassium iodide solution (not included), or alternatively, a white fissure sealant or glass ionomer cement may be used depending on cavity depth and location. In anterior teeth, an opaque resin can be considered.

- Clinical caution is necessary to avoid contact with the gingival margin. Inadequate isolation may cause discomfort or mild irritation.

- Do not apply in cases of known deep leakage.
- May cause transient discomfort in hypersensitive areas.
- In deep cavities, as leakage toward the pulp may occur, its use is discouraged unless diluted 1:10 with distilled water.

Handling Instructions:

- Close the bottle immediately after use.
- Use gloves.
- Store in a light-protected place.
- If the skin is stained, clean with household bleach and cotton, or with 10-volume hydrogen peroxide.
- In case of spills on surfaces, immediately use concentrated sodium hypochlorite.
- Do not transfer the product to another container.

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Aesthetic Warning:

Due to the silver reaction with dentin, dark staining may occur. Therefore, use of FAgamin® is not recommended on permanent anterior teeth where aesthetics are important.

Adverse Reaction:

Transient gingival irritation may rarely occur.

Composition:

38% w/v solution of silver diamine fluoride, $\text{Ag}(\text{NH}_3)_2\text{F}$.

Presentation:

Black plastic dropper bottle containing 5 ml or 1 ml.

Storage:

Store between 4 and 30 °C. Keep tightly closed and protected from light. Do not freeze. Keep away from heat sources.

Autorized by ANMAT PM 1277-14

Technical director: Miryam Della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

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