Tedequim

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**TESTPLAC® DETECTOR DE BIOFILM DENTAL
(PLACA BACTERIANA) Un tono - Dos tonos**

PÁGINA..... 2

PÁGINA..... 3

INSTRUCTIONS FOR USE

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TESTPLAC® DENTAL BIOFILM DISCLOSING SOLUTION (DENTAL PLAQUE)

One Tone - Two Tones

Dental biofilm or plaque is an invisible film, composed of bacteria that accumulates every day, especially in the interdental spaces and in the cervical region. This biofilm is the main cause of dental caries and periodontal diseases; therefore, its complete removal is important.

TESTPLAC® contains innocuous dyes that temporarily stain the biofilm or plaque in order to allow a proper brushing. The periodic use of TESTPLAC® contributes to a better oral hygiene and control dental diseases.

Instruction of Use

- 1- Wet a cotton swab with TESTPLAC® and apply on all surfaces of the teeth.
- 2- Remove the excess by rinsing with a small amount of water.
- 3- Observe the teeth with good lighting. If you use TESTPLAC® **two tones** the biofilm stained blue will indicate old plaque and the new one will appear stained red. If you use TESTPLAC® **one tone**, all plaque will be stained green or yellow. In this case, the developing action is manifested by the appearance of a fluorescent yellow color when illuminated, preferably with halogen or LED light. It has the advantage that if the plaque is not completely removed, it goes unnoticed in the mouth when the patient leaves the office.
- 4- Make a proper brushing until the color disappears.
- 5- Apply TESTPLAC® again in order to ensure the complete elimination of the bacterial biofilm.

Cautions and Warnings:

- Use gloves.
- Children can use TESTPLAC® under the supervision of an adult.

Composition:

- TESTPLAC® Green FD&C dyes at 0.8% in glycerol.
- TESTPLAC® Yellow contains sodium fluorescein 0.75% in glycerol.
- TESTPLAC® Two tones is an aqueous solution based on FD&C Green N°3 at 1% and FD&C red N°3 at 1% in a way that one does not alter the action of the other.

Presentation:

- TESTPLAC® One tone green: Dropper bottle x 25 ml.
- TESTPLAC® One tone yellow: Dropper bottle x 25 ml.
- TESTPLAC® Two tones: Dropper bottle x 10 ml.

Storage

Store between 4 and 40 °C protected from light.

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Autorized by ANMAT PM 1277-22
Technical director: Miryam Della Vedova - Pharmacist - Licence 2107
Exclusive sale to professionals and sanitary facilities

Manufactured by TEDEQUIM SRL
Bv. De los Polacos 6136- Los Boulevares
X5147GGP Córdoba - Argentina
Tel: 54-3543-448260
www.tedequim.com.ar
Rev.E

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