Tedequim

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TEDEQUIM® CLARIDENT-X PRO

PÁGINA..... 2

PÁGINA..... 3

INSTRUCTIONS FOR USE

TEDEQUIM® CLARIDENT-X PRO

PAGE..... 4

PAGE..... 5

TEDEQUIM®
CLARIDENT-X PRO

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TEDEQUIM®

CLARIDENT-X PRO

INSTANT MIXING AND DIRECT APPLICATION.

For professional use only

Read carefully before use.

Clarident X PRO is a complete dental whitening kit that includes **TEDEQUIM® Gingival Barrier**. It is designed for in-office treatments on a single patient.

The active whitening agent in **Clarident X PRO** is hydrogen peroxide, the strongest oxidizing agent among peroxides, which cleans and whitens both vital and non-vital teeth.

The effectiveness of **Clarident X PRO** is well-documented for stains caused by fluorosis, tetracycline, nicotine, certain foods, aging, etc. However, in cases of severe fluorosis or third-degree tetracycline stains, it is advisable to use a microabrasion technique with **Clarident TA**.

Instructions for use:

Make an accurate diagnosis of the origin of the tooth discoloration.

- 1- Ensure the gingiva in the treatment area is healthy.
- 2- Treat caries and any possible microleakage.
- 3- Treat sensitivity using **TEDEQUIM® Kdesin** before applying the whitening gel.
- 4- It is recommended to perform a complete prophylaxis. Do not use materials that contain glycerin or fluoride.
- 5- Isolate the gums using **TEDEQUIM® Gingival Barrier**:
 - a. Apply a 4 to 6 mm wide layer of the barrier over the gingiva. Seal the interproximal spaces. Extend the resin approximately 0.5 mm over the dry enamel to create a proper seal.
 - b. Apply the resin one tooth beyond the last tooth to be whitened.
 - c. Make sure all gingival tissue is fully covered.
 - d. Light-cure the resin for 20 seconds per guide width.
- 6- Evaluate the initial tooth shade and take a photograph.
- 7- Prepare the whitening gel as follows:
 - a. Remove the cap from the Factor B syringe and screw the connector on tightly. Then remove the cap from the Factor A syringe and attach it to the connector.
 - b. Pass the contents of Factor A into Factor B, and then vice versa. Repeat the transfer 6 to 8 times until fully homogenized.
 - c. Finally, remove the connector and the empty syringe.
- 8- Apply **Clarident X PRO** using the applicator tip over the teeth to be treated, with a layer 2 to 3 mm thick.

Do not use metal spatulas.

- 9- Let the gel act for approximately 40 minutes. If necessary, repeat the procedure using the remaining product.

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- 10-Suction the gel, rinse thoroughly with water, and remove the gingival barrier.
Evaluate the final shade and take a photo for record.
- 11-The number of sessions will depend on the severity of the case. After the final session, apply TEDEQUIM® Kdesin or NEUTRODENT, desensitizers based on potassium nitrate and fluoride.
- 12-Advise the patient not to eat for at least one hour. Any sensitivity should resolve within one to two days.
- 13-Recommend that the patient follow a white diet after the whitening

Recommendations

Wear gloves.

During the procedure, the enamel becomes especially permeable. Patients should avoid smoking or consuming staining foods or beverages. If the treatment was performed in a single session, the white diet should be followed for at least two days.

If the product has been stored in the refrigerator, let it reach room temperature before use.

Precautions

Avoid contact with skin or mucous membranes. If contact occurs, rinse thoroughly with water for several minutes or cover the affected area with **TEDEQUIM® Gingival and Lip Protector**

Contraindications

- Do not use on patients with dentin hypersensitivity.
- Do not apply to dentin or carious teeth.
- Do not use in areas with potential microleakage.

Contents

- Factor A: Hydrogen peroxide, 1.5 ml
- Factor B: Gel, 0.5 g
- TEDEQUIM® Gingival Barrier: 1 syringe x 1.2 ml
- Accessories

Storage conditions

Store between 4 and 20 °C.

Autorized by ANMAT PM 1277-29

Technical director: Miryam Della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

Manufactured by TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba - Argentina

Tel: 54-3543-448260

www.tedequim.com.ar

Rev.A

Tedequim

+54 3543 448260

info@tedequim.com

www.tedequim.com

Tedequim SRL

Bv. de los Polacos 6136, X5147GGP

Córdoba, Argentina