Tedequim

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TEDEQUIM®

CLARIDENT X

INSTANT MIXING AND DIRECT APPLICATION.

For professional use only

Read carefully before use.

Clarident X is an in-office dental bleaching kit intended for single-patient treatment.

The whitening agent in Clarident X is hydrogen peroxide, the most powerful oxidizing agent among peroxides, providing effective cleaning and whitening of both vital and non-vital teeth.

The effectiveness of Clarident X is well-established for stains caused by fluorosis, tetracycline, nicotine, certain foods, aging, etc. However, in cases of severe fluorosis or third-degree tetracycline staining, it is recommended to use a microabrasion technique with Clarident TA.

Instructions for use:

Perform an accurate diagnosis of the origin of the discoloration.

- 1- Ensure the gingiva in the treatment area is healthy.
- 2- Treat caries and any possible microleakage.
- 3- Treat hypersensitivity with TEDEQUIM® Kdesin before applying the whitening gel.
- 4- It is advisable to perform a complete prophylaxis. Do not use materials containing glycerin or fluoride.
- 5- Isolate the gingiva using TEDEQUIM® Gingival Barrier.
- 6- Evaluate and photograph the tooth shade before the procedure.
- 7- Prepare the whitening gel as follows:
Remove the cap from the Factor B syringe and securely attach the connector.
Then remove the cap from the Factor A syringe and connect it to the other end of the connector.
Pass Factor A into the Factor B syringe and vice versa.
Repeat the exchange process 6 to 8 times until homogenized.
Finally, remove the connector and the empty syringe.
- 8- Apply Clarident X using the applicator tip onto the teeth to be whitened, with a thickness of 2 to 3 mm.
Do not use metal spatulas.
- 9- Leave the gel on for approximately 40 minutes. If necessary, repeat the procedure using the remaining product.
- 10- Suction the gel, rinse thoroughly with water, and remove the gingival barrier.
Evaluate and photograph the tooth shade.
- 11- The number of sessions will depend on the severity of the case.
At the end of the sessions, apply TEDEQUIM® Kdesin, a desensitizing agent based on potassium nitrate and fluoride.
Allow it to act according to the patient's level of sensitivity, then remove the excess.
- 12- Advise the patient not to eat for at least one hour. If the patient

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experiences sensitivity, it will subside in one or two days.
13-Recommend a white diet after the whitening treatment.

Additional Recommendations:

Use gloves during the procedure.

During treatment, the enamel becomes especially permeable. Patients must be warned to avoid smoking or consuming staining foods or drinks.

If the treatment is performed in a single session, the white diet should be followed for at least two days.

If the product has been refrigerated, allow it to reach room temperature before use.

Precautions:

Avoid contact with skin or mucosa. If contact occurs, rinse thoroughly with water for several minutes or cover the area with TEDEQUIM® Gingival and Lip Protector.

Contraindications:

- Do not use in patients with dentin hypersensitivity.
- Do not apply to dentin or carious teeth.
- Do not apply in areas with possible microleakage.

Contents:

- Factor A: Hydrogen peroxide, 1.5 ml
- Factor B: Gel, 0.5 g
- KDESIN: Desensitizing agent based on potassium nitrate and sodium fluoride, 2 ml
- Accessories

Storage conditions:

Store between 4 and 20 °C.

Autorized by ANMAT PM 1277-29

Technical director: Miryam Della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

Manufactured by TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba - Argentina

Tel: 54-3543-448260

www.tedequim.com.ar

Rev.C