Tedequim

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www.tedequim.com

Rev. A

ACIDENT
Topical Acide Gel
Topical gel of fluoride ions 1.23%.

Indications:

Topical gel for caries prevention.

The presence of fluoride makes the enamel or dentin more resistant to acid dissolution and therefore less susceptible to caries. This product is great for the prophylaxis of dental caries.

Sodium fluoride gels at acid pH are quick to apply.

Instructions for use:

1. Remove bacterial plaque, dry the tooth surface with air.
2. Choose a disposable bucket according to the size of the patient's mouth.
3. Fill 1/3 of the bucket with gel and apply for 1 minute.
4. Remove the bucket and ask the patient to remove the excess gel, without washing.
5. Warn the patient not to eat or drink for 30 minutes.
6. The gel can also be used with a brush or spatula. Always leave it act for at least one minute.

Recommendations:

Applications for preventive purposes should be repeated every six months. In case of greater cariogenic activity it is convenient to perform the topicalization every three months.

Use disposable buckets.

Warnings:

If after the application the patient manifests allergic reaction in gum, the gel can be eliminated with rinses.

- Do not use acid gels with fluorides in patients with restorations.
- Do not apply in patients with:
 - Ulcerative gingivitis.
 - Stomatitis.

Care in the manipulation:

Do not place excess gel in the bucket so that it overflows.

Keep in a cool place.

Keep of the reach of children.

Composition:

Fluoride ions 1.23%, Phosphoric acid c.s.p pH: 3.0 - 3.5, excipients c.s. p 100%.

Storage:

Keep between 15°C and 30 °C. Do not store near heat sources

Autorized by ANMAT PM 1277-21

Technical director: Miryam della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

Manufactured by TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba - Argentina

Tel: 54-3543-448260

www.tedequim.com.ar

Rev. A