Tedequim

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TEDEQUIM® PROPHYLAXIS POWDER

The product is a powder based on sodium hydrogen carbonate (sodium bicarbonate) which has a micro particle size < 74 microns. In combination with air and water enables effective cleaning of the oral cavity without abrasion of the enamel. It has a pleasant lemon flavour which is well tolerated by patients. This powder inhibits the acid activity of bacterial plaque by alkalising the medium. It also removes bacterial plaque from the gingival sulcus, soft particulate deposits in the interproximal area and allows the professional to work in a hygienic mouth.

This mixture of sodium bicarbonate, air and water forms a precise jet, which is more effective than conventional brushes, calyx-shaped rubbers, polishing pastes and strips or abrasives.

The fluidity, the particle size and the hygrophobicity make this powder exclusively designed for use in dental prophylaxis equipment, in combination with pressurized air and water flow. These devices work as a high-speed handpiece and are connected to a pneumatic outlet on the dental chair stage. They are designed for use on anterior and posterior teeth.

Instructions for use:

Install the dental prophylaxis equipment in a pneumatic output of the dental chair's platen. Follow the instructions of the equipment manufacturer (pressure control - between 3 and 4 Kg/cm² - and water flow). Then place the required amount of powder in the reservoir of the unit according to the instructions of the equipment. Activate the system and clean and sanitise the patient's mouth at a distance between 3 mm and 5 mm from the exit of the instruments and the tooth. Apply at an angle between 30° and 60° to the tooth surface in a circular motion.

Composición:

Sodium hydrogen carbonate 98%, sodium saccharin, silica colloid, lemon essence, 2 %.

Presentation:

Polyethylene container x 100 g.

Storage conditions:

Keep tightly closed preferably at temperatures between 4 °C and 30 °C

Autorized by ANMAT PM 1277-28

Technical director: Miryam Della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

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