Tedequim

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www.tedequim.com

Rev. B

TESTPLAC®
BACTERIAL PLAQUE DISCLOSING SOLUTION
Single color - Dual color

Bacterial plaque is an invisible film, composed of bacteria that accumulates every day, especially in the interdental spaces and in the cervical region. This plaque is the main cause of dental caries and periodontal diseases, therefore its complete removal is important.

TEDEQUIM® contains innocuous dyes that temporarily stain the plaque in order to allow a proper brushing. The periodic use of TEDEQUIM® contributes to a better oral hygiene and control dental diseases.

Instruction of Use:

- 1) Wet a cotton swab with TEDEQUIM® and apply on all surfaces of the teeth.
- 2) Remove the excess by rinsing with a small amount of water.
- 3) Observe the teeth with good lighting. If you use TEDEQUIM® **Dual color**, the plaque stained blue will indicate old plaque and the new one will appear stained red, otherwise, the whole plaque will be stained green or yellow.
- 4) Make a proper brushing until the color disappears.
- 5) Apply TEDEQUIM® again in order to ensure the complete elimination of the bacterial plaque.

Cautions and Warnings:

- Use gloves.
- Children can use TEDEQUIM® under the supervision of an adult

Composition:

TEDEQUIM® Dual color is an aqueous solution based on FD&C Green N° 3 at 1% and FD&C red N° 3 at 1% in a way that one does not alter the action of the other.

TEDEQUIM® Yellow contains sodium fluorescein 0.75% in glycerol.

TEDEQUIM® Green FD&C dyes at 0.8 % in glycerol.

Presentation:

Single color - Green: It dyes the bacterial plaque green.

Dropper bottle x 25 ml.

Single color - Yellow: Disclosing action manifests itself with the appearance of a fluorescent yellow color, when it is illuminated, preferably with halogen light or LED. It has the advantage that if the plaque is not completely removed, it goes unnoticed in the mouth when the patient leaves the office.

Dropper bottle x 25 ml.

Dual Color: It has the particularity that it is able to distinguish the old, thicker plaque from the new, recent and thin plaque. It dyes blue the first and red the second one.

Dropper bottle x 10 ml.

Storage:

Keep between 4°C and 40 °C protected from light.

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