



Tedequim

**INSTRUCCIONES DE USO
TEDEQUIM® ÁCIDO GRABADOR
PÁGINA..... 1**

**INSTRUCTIONS FOR USE
TEDEQUIM® ETCHING ACID
PAGE..... 2**

TEDEQUIM®
ACIDO GRABADOR
Ácido ortofosfórico 37% - gel azul

Instrucciones de uso:

Para la técnica de grabado ácido sobre esmalte, limpiar y secar la superficie antes de la aplicación. Aislar la superficie. Agitar y colocar la aguja aplicadora en la jeringa. Aplicar únicamente en la zona del esmalte donde se vaya a recubrir con material restaurador posteriormente. Dejar actuar entre 15 y 30 s. Luego enjuagar con abundante agua. Sacar la aguja aplicadora y guardar la jeringa bien tapada.

Se deja a criterio del profesional la utilización en dentina.

Contenido:

Jeringa por 7 g

Jeringa por 12 g

Aplicadores plásticos

Almacenamiento:

Conservar bien tapado y protegido de la luz a temperatura entre 15° y 30°

Autorizado por ANMAT PM 1277-1.

Venta exclusiva a profesionales e Instalaciones sanitarias

Director técnico: Miryam Della Vedova- Matrícula N° 2107

Elaborado por TEDEQUIM SRL

Bv. De los Polacos 6136- 5147 Córdoba- Argentina

Tel: +54 3543 448260

www.tedequim.com

Rev. B

ETCHING ACID
37% Orthophosphoric acid in gel
For Professional Use Only

Instructions for use:

The surface must be cleaned and dried before the application. Isolate the surface. Shake the syringe and remove the cap in order to attach the disposable plastic needle to the end of the syringe. Apply gel only on enamel surfaces to be covered with restorative material. Allow ETCHING ACID to remain in place for 15 to 30 seconds and then rinse thoroughly with water. Detach the disposable plastic needle and place the original cap tightly to avoid evaporation.

The use of this product on dentin is left up to the professional.

Content:

Syringe of 7 g
Syringe per 12 g
Plastic applicators

Storage:

Store tightly covered and protected from light at a temperatura between 15° and 30°.

Aprobado por ANMAT 1277-1

Dirección Técnica: Miryam Della Vedova - Farmacéutica- Mat. 2107

Exclusive sale to professionals and sanitary facilities

Made in Argentina

Manufactured by TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba- Argentina

Tel: 0054 3543 448260

www.tedequim.com.ar

Rev. B