Tedequim

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Rev. B

HEMODEN

Aluminum potassium sulfate hemostatic solution

Water-soluble aluminum products are both hemostatic and astringents. HEMODEN is a local action agent, an effective astringent that causes the contraction of tissue by settling down the blood proteins and creating a coagulum. When applying to the mucous membrane slightly swollen and edematous, it reduces the membrane permeability of capillary cellules. Thus, the movement of plasmatic proteins along the capillary ceases and the local edema, swelling and tissue fluids are reduced.

Astringents action is temporary and it only takes place on the tissue surface where it is directly in contact with; it doesn't penetrate (inside) the cellules.

Indications:

Its main use in dentistry is to cease or stop the capillary hemorrhage and reduce the mucous membrane inflammation.

Instructions of use:

A cotton pellet soaked in HEMODEN is rubbed (with moderate pressure) against the cut tissue. After waiting for astringent to take action, the area is cleaned with air/water spray. If bleeding persists, repeat until hemostasis is achieved.

Cautions, Warnings and Recommendations:

Aluminum solutions are acid and can produce irritation over soft tissues.

- Wear gloves.
- Do not eat.
- In the case of acute hemorrhages, the application of this product may not be enough and other hemostats or constrictor vessels may have to be used.
- If the patient shows burning, irritation, it can be eliminated simply by rinsing with plenty of water. . This can occur because aluminum solutions are acidic and can irritate soft tissues in some cases.
- Do not apply in patients with:
 - Allergy.
 - Ulcerative gingivitis.
 - Stomatitis.
 - Acute bleeding.
- Keep it out of the reach of children.

Storage conditions:

Keep between 4° and 30° C in closed bottle.

Content:

20 ml.

Active ingredient:

Potassium and aluminum sulfate

Autorized by ANMAT PM 1277-15

Technical director: Miryam della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

Manufactured by TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba - Argentina

Tel: 54-3543-448260

www.tedequim.com.ar

Rev. B