**Tedequim**

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**TEDEQUIM® BACTEROL**

**PÁGINA..... 2; 3; 4**

**INSTRUCTIONS FOR USE**

**TEDEQUIM® BACTEROL**

**PAGE..... 5; 6; 7; 8**

TEDEQUIM®  
BACTEROL

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**TEDEQUIM®**  
**BACTEROL**

**General Purpose Cleaner and Disinfectant Solution**

Action against a broad spectrum of bacteria, fungi and virus.

Effective against Coronavirus

- Surface cleaner and disinfectant.
- Ultrasonic or manual cleaning and disinfectant of instruments
- Disinfection of hydric circuit of the dental units.

BACTEROL is a cleaning and disinfectant solution for general use in Hospitals, clinics, veterinarians, offices, etc.

It can be applied undiluted on surfaces such as floors, countertops, dental office chair, spittoons, stretchers, etc.

It is also used diluted to:

1) Wash and disinfect instruments and equipment for medicine, surgery, dentistry, veterinary medicine, etc. which will later be rinsed and subjected to sterilization in a dry oven or autoclave. This solution does not sterilize and must be replaced once used.

2) Use for the sanitization of water circuit in dental units.

Properties: Quaternary ammoniums enhanced with isopropyl alcohol are effective against Staphylococcus Aureus; Salmonella Choleraesuis; Escherichia Coli; Pseudomonas Aeruginosa; Trichophyton Mentagrophytes; Candida Albicans; Mycobacterium Smegmatis; HIV virus; SARS\_COV2 virus; Herpes Simplex; Mold and Moisture.

Stability of the product in its primary container:

The product must be kept in its original container, well closed. It should not be transferred to another container to avoid oxidation processes. Bacterol is incompatible with anionic detergents such as soaps and alkyl benzene sulfonates.

Collateral or secondary actions:

Not known so far.

\*In the case of accidental ingestion or discomfort, go to the doctor, to the healthcare center closest to your address.

Composition:

Quaternary ammonia, EDTA Na<sub>2</sub>.2 H<sub>2</sub>O, Isopropyl alcohol, sodium hydroxide, other excipients and water. It does not contain sulfates, does not stain or oxidize.

Conditions of conservation, storage and transport:

Keep tightly closed at room temperature, preferably between 4 to 30°C. Do not store near heat sources.

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### Instructions for use: (product preparation)

- **Surfaces:** For cleaning and disinfection, apply undiluted BACTEROL directly on the surface to be treated, wetting the entire area. Leave to act for 2 minutes (For action against Tuberculosis, leave to act for 10 minutes at 20°C). Then run a dry cloth or paper towel over the surface to remove the product. Discard the used towel.

- **Instrumental:**

- **Ultrasonic cleaning:** Dilute 30 ml of Bacterol per liter with water in the basin for ultrasonic washing. Immerse previously rinsed instruments under running water to remove visible residues and activate ultrasound for 5 minutes or the time suggested by the ultrasound manufacturer.

Then remove the objects, rinse with plenty of running water. Change the solution in the pool once a day or as required.

- **Manual cleaning:** Rinse used instruments under running water to remove visible deposits. In a suitable container dilute 30 ml of Bacterol per liter of wash water and immerse the instruments in that solution. Scrub with a brush or sponge, leave to act for about 10 minutes and then rinse with plenty of water.

- **Hygienization of the water circuit of the dental unit:**

Place 30 ml of BACTEROL per liter of water in the tank of the instrument disinfection system of the stage. Circulate on the rotating instruments. Then put distilled water in the tank and use the equipment.

### Instructions for the disposal of the used product:

The solution after being used is discarded by the common sanitary network. The empty container must not be reused. This container is discarded in the recycling container for plastic elements.

### Restrictions on the materials to be treated by the product:

It is applied on plastic surfaces (polycarbonates, polyvinyl chloride, polypropylenes, polystyrene), stainless steel, painted surfaces, flexiglas, glass and other hard non-porous surfaces

### Precautions and warnings about its use:

- FORBIDDEN ITS FREE SALE.
- CAREFUL! Irritating to the eyes.
- Do not mix with other products.
- Use Gloves for its application.
- Do not eat.
- Do not apply on food and animals
- KEEP OUT OF THE REACH OF CHILDREN AND DOMESTIC ANIMALS.
- In case of contact with eyes and skin, wash for 15 minutes. If irritation persists, consult a doctor, taking the product container or label.
- In case of accidental ingestion, do not induce vomiting. Consult a doctor immediately, taking the product container or label.

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- Keep the product in its original container
- Do not reuse the containers

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Exclusive sale to professionals and sanitary facilities

Made in Argentina

Manufactured by : TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba- Argentina

Tel: 54-3543-448260

[www.tedequim.com.ar](http://www.tedequim.com.ar)

Rev. C